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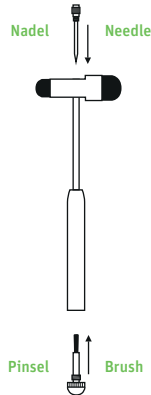
Zhejiang Honsun Medical Instrument Co., Ltd.
Sanjiang Street, Building 8 of Art & Crafts Park Beiao
Industrial Estate, 325100 Wenzhou, Zhejiang, P.R.China
Mfg. Date:

Please read the instructions thoroughly and carefully before using this product.

Use/safety: The reflex hammer should only be used as a non-invasive medical instrument to elicit neural responses on healthy skin. Due to its functions, this instrument contains a punctured spike that may cause serious injury if used incorrectly or for other purposes. The reflex and percussion hammer may only be used by authorized professionals. Keep the instrument out of the reach of children.

Intended use: Diagnostic instrument for the external triggering of myotatic, plantar, and cutaneous reactions. Any use other than that described here is not in accordance with the intended use of the instrument. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Operation: Knock softly from the wrist with the reflex hammer on the corresponding attachment tendon. This results in a reflex contraction (contracting) of the muscle concerned. The needle and brush can be used to test the various skin/surface sensitivities.

Manual cleaning: Wipe reflex hammer clean with a moistened tissue or surface cleaning agent.

Disinfection: Clean the reflex hammer thoroughly before each use. Use only a clean cotton cloth or swab and an isopropyl alcohol solution to clean all contaminated parts. The instrument may not be sterilized. It is imperative to prevent the contaminants from drying on the material. Disinfection aims to protect both the staff and the patient during use.

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80, 20537 Hamburg, Germany
SRN: DE-AR-000000001
Art. No.: 5810033 Manufacturer No.: HS-401G3

MDD 93/42/EEC

EN 980:2008; EN 1041:2008;
EN ISO 13485:2016; EN ISO 14971:2007

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WESTMED distributed by Fleischhacker GmbH & Co. KG