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 HONSUN (NANTONG) Co., Ltd  
No. 8 Tongxing Road Economic  
& Technological Development Area,  
226009 Nantong City,  
Jiangsu Province, P.R. China

 Shanghai International  
Holding Corp. GmbH (Europe)  
Eiffestraße 80, 20537 Hamburg  
Germany, SRN: DE-AR-000000001

Please read these instructions for use carefully and completely before use, and observe the care instructions.

The tourniquet is used for blood stanching for intravenous puncturing of vessels for the purpose of blood sampling or the insertion of intravenous access. In addition, the tourniquet can also be used in an emergency to stop acute bleeding. Observance of all the information in these operating instructions and the relevant regulations and guidelines is part of the intended use. Any other use is not as intended, and the manufacturer is not liable for any resulting damage!

**This product may only be used by authorized specialist personnel!**

**Intended patient group:** Adults, children, infants, and babies.

### Safety instructions

- Keep the tourniquet out of reach of non-medically qualified persons.
- Check the complete tourniquet before use to ensure that it is not defective.
- Do not use the product near or directly on the neck. Such use is associated with a fatal risk of strangulation and must be avoided at all costs.

- Do not position the closure directly on a pacemaker and never use it in an MRI.
- Before using the device for the first time, and after each use, it should be cleaned to prevent germ colonization or germ transmission. In case of contamination with secretions, other body fluids, or toxic substances, please follow the internal hygiene regulations and, in case of doubt, dispose of as special waste.
- Automatic cleaning or disinfection is not possible. Please follow the cleaning and disinfection instructions.
- Immediately after carrying out the medical treatment, please open the cap as quickly as possible by pressing the multifunction clip. Avoid uncontrolled opening.
- Dispose of the tourniquet if there is visible wear on the band/closure.

### Handling

- Position the tourniquet at the desired point on the limb. The "click" when closing the multifunction clip signifies a secure connection. Adjust the desired tourniquet pressure by tightening the opposite end of the strap. Avoid pinching or squeezing the skin. To release the tourniquet immediately, press the top release button on the multifunction clip.

### Cleaning and disinfection

Please follow your internal hygiene procedures to reduce the risk of cross-contamination. After application to a patient's skin, the tourniquet should be cleaned and disinfected with a 70% alcohol solution (e.g., with soaked wipes). Observe the exposure times of the product used.

### Disposal

The product must be disposed of in accordance with local regulations.

### Storage and transport

The tourniquet must be stored and transported in such a way that it is protected from dust, moisture, and contamination.

### Maintenance and service

The end of product life is normally determined by wear and damage from use. Damaged parts must be replaced! For possible repairs, please contact your dealer.

### Warranty

If the product is handled correctly and our instructions for use are observed, the statutory warranty of two years commencing on the date of sale applies.

All information without guarantee – subject to change.

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