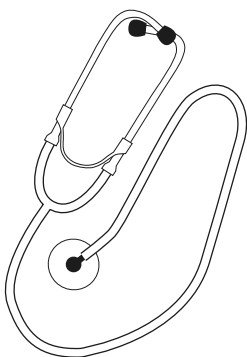
WESTMED

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Please read these instructions for use carefully and completely before use and observe the care instructions.

The stethoscope is a diagnostic instrument and consists of an earpiece, a tube system, and the chest piece. It is used to auscultate (listen to) heart sounds, lung sounds, and other audible vital parameters. Depending on the examination, the stethoscope is to be placed on the desired body region and used under slight pressure. Observance of all the information in these operating instructions and the relevant regulations and guidelines is part of the intended use. Any other use is not as intended and the manufacturer is not liable for any resulting damage!

This product may only be used by authorized specialist personnel!

Intended patient group

Adults

Safety instructions

- Before each use, check the stethoscope for proper function and cleanliness. Do not use the product if you notice any damage or soiling.
- Never fit the stethoscope without earpieces to avoid injury to the ear canal! Damaged earpieces must be replaced before use.
- Always ensure that the new parts are firmly seated after replacement. Only use original spare parts and accessories.

- Never place the stethoscope in liquids.
- The stethoscope is not suitable for sterilization processes!
- Ambient noise can interfere with use.
- Protect the stethoscope from high temperatures, strong solvents, and oils.

Operation

- First, place the binaural earpieces in the ears.
- To check that they are working properly, gently tap the diaphragm with your finger.
- If you do not hear a tapping sound, there is a defect.
- If you hear a tapping sound, you can start the examination.

Cleaning and disinfection

After each patient, the stethoscope should be cleaned and disinfected with a 70% alcohol solution (e.g., with wet wipes). Always observe the exposure times of the product used. Furthermore, the individual components of the stethoscope should be cleaned thoroughly on a regular basis. Use a 70% alcohol solution or a mild soap solution for this purpose. The earpieces and the membrane can be removed for thorough cleaning. To clean the membrane, screw off the retaining ring. Handle the diaphragm carefully to avoid damage! Do not reassemble the stethoscope until all parts are dry.

Disposal

The product must be disposed of in accordance with local regulations.

Storage and transport

Store and transport the stethoscope protected from dust, moisture, and contamination.

Maintenance and service

The end of product life is normally determined by wear and damage from use. Damaged parts must be replaced! For possible repairs, please contact your specialist dealer.

Warranty

If the product is handled properly and our instructions for use are observed and original parts are used, the statutory warranty of two years commencing on the date of sale applies.

Art. No.: 5810009
Manufacturer No.: HS-30A

MDR 2017/745

ISO 10993-5:2009; ISO 10993-10:2010;
ISO 15223:2016; ISO 14971:2012;
ISO 13485:2016

WESTMED distributed by
Fleischhacker GmbH & Co. KG
An der Silberkuhle 18, 58239 Schwerte, Germany
Tel.: +49 2304 9310, Fax: +49 2304 931199
info@fleischhacker.biz

HONSUN (NANTONG) Co., Ltd
No. 8 Tongxing Road Economic & Technological
Development Area, 226009 Nantong City,
Jiangsu Province, P.R. China

Shanghai International Holding Corp. GmbH
(Europe), Eiffestraße 80, 20537 Hamburg,
Germany, SRN: DE-AR-000000001