Accutrend Control TG 1

REF	CONTENT	SYSTEM
11538152190	1.5 mL	Accutrend GCT Accutrend Plus

English

Intended use

For performance checks of the Accutrend GCT or Accutrend Plus meter using Accutrend Triglycerides test strips.

Suitable for self-testing.

Summary

Use Accutrend Control TG 1 solution to check the performance of your Accutrend GCT or Accutrend Plus meter and the Accutrend Triglycerides test strips and their correct handling.

Perform a performance check in the following situations:

- when learning how to measure triglycerides
- after changing the batteries in the meter
- after opening a new pack of test strips and re-calibrating the meter
- after cleaning the meter
- if you have reasons to doubt the validity of a reading

How the test works

Each test strip has a test area containing detection reagents. When control solution is applied, a chemical reaction takes place and the test area changes colour. The meter records this change in colour and converts the measurement signal to the displayed result using the data previously entered via the code strip.

Important notes

Do not perform performance checks at temperatures below 18 °C or above 30 °C. Allow test strips and control solution just removed from the refrigerator to reach a temperature within this range before use. Use a fresh test strip if the drop of solution has not completely covered the test area, if several drops were applied or if a drop contained air bubbles. Do not touch the tip of the bottle.

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Safety data sheet available for professional user on request.

All components of the pack can be discarded along with household waste or, if used in a laboratory or doctor’s office, according to relevant local guidelines.

Storage and stability

Store at 2-8 °C up to the stated expiration date. Do not freeze. When stored at the conditions stated above, opened bottles of triglycerides control solution are stable for 3 months but not beyond the expiry date given on the pack. When opening the bottle for the first time, write the date onto the bottle label.

Sources of error

If a performance check with the Accutrend Control TG 1 solution produces a reading outside the stated range, please verify:

- that the triglycerides measurement was carried out correctly (see operator’s manual)
- that the test strip guide is clean
- that the test strips and the control solution used were correctly stored (expiry date, storage conditions)
- that meter, test strips and control solution were at the correct temperature of 18 °C to 30° °C during the measurement.

Materials provided

- 1 bottle of Accutrend Control TG 1: contains 1.5 mL of Accutrend Control TG 1 solution; sufficient for approx. 30 performance checks.

Materials required (but not provided)

- One of the above listed meters with operator’s manual
- Accutrend Triglycerides test strips
- Paper towel

Preparing and performing the measurement

- Please read the information in this Method Sheet and in the operator’s manual for your meter.
- Carry out a performance check like you would do for a triglycerides measurement. Note: use Accutrend Control TG 1 solution instead of blood.
- Remove one Accutrend Triglycerides test strip from the container and apply one drop of control solution to the test pad, moistening it completely.
- Hold the tip of the dropper bottle vertically downward and dispense one drop from a height of about 1 cm, without touching the test pad with the tip. Wipe the tip of the dropper bottle before and after use with a clean paper tissue or swab, and close the bottle tightly afterwards.
- Compare the result you get with the value sheet provided with the test strip. If the result is within the stated range, it shows that the meter and test strips are performing OK.
- If the control result is outside the stated range, check that you have followed the correct procedure (see section “Sources of error”). After eliminating all potential sources of error, repeat the performance check. If the control result is still outside the stated range after the second check, please contact your Roche Diagnostics representative.

Instructions for healthcare professionals

Applications: Perform quality control steps according to guidelines established for your practice or laboratory.

Reagents

Triolein: 280 mg/dL

Specific performance data

Reproducibility (day-to-day imprecision):

In a typical measurement series, the majority of the measured coefficients of variation (CV) was < 3.1 %.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

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